English

HUMAN URINE PROGRAMME: RQ9115

CONFIRMATION OF KIT CHARACTERISTICS AND RECEIPT DATE

Please confirm that the correct number of samples are present and that your samples have the appearance as indicated in the CHARACTERISTICS section below. Please confirm that none of the vials are broken and notify your local Randox representative immediately if there are any discrepancies. Finally, please log on to www.riqas.net to confirm the exact date on which you received this kit.

CHARACTERISTICS

The pack contains 24 vials of lyophilised 10 ml samples. The vials are labelled with the sample numbers 1 to 12. Note that there are 2 x 10 ml vials for each sample number.

PREPARATION/STORAGE/STABILITY OF SAMPLES

The vial is sealed under vacuum. Open the vial very carefully, avoiding any loss of material and using a **calibrated pipette** reconstitute with an accurately measured **10 ml volume** of freshly double distilled water at **+20°C to +25°C**. Replace the rubber stopper, close vial and leave to stand for 30 minutes out of bright light before use. Ensure that the lyophilised contents are completely dissolved by swirling gently. Do not shake the vial. Please analyse the sample within 2 days of reconstitution in the absence of bacterial contamination, with the exception of the analytes discussed below. The samples should be stored at 2 - 8°C when not in use.

Catecholamines, Metanephrines and Vanillylmandelic Acid (VMA) and Oxalate: These analytes are unstable in the reconstituted urine sample. Thirty minutes after complete reconstitution of urine sample take one 3 ml aliquot and add one (1) drop of HCl from HCl dropper bottle. For oxalate measurement, add EDTA to the urine sample at a concentration of 5 mg/10 ml material. This is to prevent the precipitation of Calcium Oxalate.

5-Hydroxyindole Acetic Acid (5-HIAA): This analyte is also unstable in the reconstituted urine sample. Thirty minutes after complete reconstitution of urine sample take one 2 ml aliquot and add one (1) drop of acetic acid from acetic acid dropper bottle. Acetic acid may be solid at +4°C. To ensure complete liquidity of acid please ensure the dropper bottle contents are brought to room temperature prior to use. **Please note that if Nitroso-Naphthol method is used for 5-HIAA, one drop of HCl from HCl dropper bottle should be added to a 2 ml aliquot of reconstituted urine. The addition of HCl is also required where 5-HIAA is assayed using HPLC methods with prior extraction.**

For all other analytes treat and use as a patient urine sample.

Vitros Analysers (Recommendations by Ortho-Clinical Diagnostics):

1. Do not pre-treat the reconstituted fluid.
2. Ensure that the correct dilution and diluent are used (see method sheet) and that samples are well mixed.
3. Make sure that the sample is run using the urine calibration on the system.
4. No adjustment to results is carried out.

SAFETY

The samples contain pooled human urine and should be handled with the same precautions used for patient specimens, which may contain potentially infectious agents. For **IN VITRO** use only.

* **IMPORTANT NOTE:** Results must arrive at **RIQAS** by **17:00 HRS GMT** on the **FINAL DATE**. If the **RECOMMENDED ANALYSIS DATE** gives insufficient time, we suggest that the sample is analysed earlier to ensure you meet the deadline. Late results will not be accepted after the final date for the next sample.

HUMAN URINE PROGRAMME / PROGRAMME CHIMIE URINAIRE / PROGRAMMA PER URINE UMANE / PROGRAMA DE ORINA HUMANA / PROGRAMA DE URINA HUMANA / HUMAN URINPROGRAM / HUMAN URIN PROGRAM / ΠΡΟΓΡΑΜΜΑ ΑΝΘΡΩΠΙΝΩΝ ΟΥΡΩΝ / PROGRAM OZNACZANIA MOCZU LUDZKIEGO / 人类尿液分析程序 / İDRAR KİMYA PROGRAMI		
RETURN OF RESULTS / DATES de RETOUR / INVIO DEI RISULTATI / ENVIO DE RESULTADOS/ PLAN FOR INSENDELSE AF / RESULTATRETUR/ ΚΥΚΛΟΣ ΕΠΙΣΤΡΟΦΗΣ / SONUÇ GÖNDERİM TARİHLERİ / ODOSLANIE VÝSLEDKOV		
CYCLE 63 / CICLO 63 / RESULTATER 63 / CYKEL 63 / ΑΠΟΤΕΛΕΣΜΑΤΩΝ 63 / CYKL ZWROTU WYNIKÓW 63 / 返回结果周期 63 / DÖNEM 63 / CYKLUS 63		
SAMPLE NO / ECHANTILLON NO. / N. CAMPIONE / N° DE MUESTRA / AMOSTRA N°. / PRØVENR. / ΑΡ. ΔΕΙΓΜΑΤΟΣ / PROV NR. / NR PRÓBK / 样品号 / ÖRNEK NO / ČÍSLO VZORKY	RECOMMENDED ANALYSIS DATE/ DATE RECOMMANDE POUR L' ANALYSE/ DATA DI ANALISI RACCOMANDATA/ FECHA DE ANALISIS RECOMENDADA/ DATA RECOMENDADA PARA ANÁLISE/ ANBEFALET ANALYSEDATO/ REKOMMENDERAT ANALYSDATUM/ ΣΥΝΙΣΤΩΜΕΝΗ ΗΜΕΡΟΜΗΝΙΑ ΑΝΑΛΥΣΗΣ/ ZALECANA DATA WYKONANIA OZNACZENIA/ 推荐分析日期 / ÖNERİLEN ANALİZ TARİHİ / ODPORÚČANÝ DÁTUM ANALÝZY	* FINAL DATE/ *DATE LIMITE/ * DATA FINALE/ *FECHA FINAL/ * DATA FINAL/ * SLUTDATO/ * SLUTDATUM/ *TEΛΙΚΗ ΗΜΕΡΟΜΗΝΙΑ/ * DATA KOŃCOWA/ *最终日期 *SON SONUÇ GÖNDERİM TARİHİ/ * FINÁLNY DÁTUM
1	10.07.23	17.07.23
2	24.07.23	31.07.23
3	07.08.23	14.08.23
4	21.08.23	28.08.23
5	04.09.23	11.09.23
6	18.09.23	25.09.23
7	02.10.23	09.10.23
8	16.10.23	23.10.23
9	30.10.23	06.11.23
10	13.11.23	20.11.23
11	27.11.23	04.12.23
12	11.12.23	18.12.23

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